



Transforming Lives®

How to Use The NuStep® T5 WellGrip™ Handle Gripping Accessory

The left hand grip is shown; the right hand grip works in a similar manner.

1 Slide WellGrip onto handlebar.

Lock WellGrip key into numbered slot of desired wrist position.

Wrap #1 strap around handlebar base to secure.



2 Insert user's hand through strap #2 and onto handlebar.

Snug strap 2. It should be secure, but does not have to be tight to be effective.



3 Wrap finger flap around fingers and secure strap #3. It should be secure, but does not have to be tight to be effective.



Materials:

Polyester, Nylon, Buna N Rubber, Latex Free

Care Guide:

Machine Wash Cold, No Bleach, Tumble Dry Low



CAUTION

Hand restraint.

User may not be able to remove hand once installed.

Supervise use of product.