



Transforming Lives®

How to Install the Accessory Bag (Rev A) on the NuStep® T5 / T5^{XR}

Tools Required

3 mm hex key
8 mm wrench (T5 only)

Accessory Kit Parts

(1) 50748 Accessory Bag, Rev A
(3) 50704 Screw, Button M5x.8x16
(1) 50605 Nut, M5x.8

- 1 Remove only the top two screws from back of seat.

If torso belt kit is already installed remove it at this time (far right).



- 2 Place bag into position and reinstall top two screws through bag holes. Tighten screws.

To reinstall torso belt kit, reattach it on top of bag (far right).




- 3 Install two new screws in middle holes of bag and into cover. Tighten screws.



- 4 For T5, remove bottom two screws from cover, and use screw and nut from kit to attach lowest third hole of bag (shown).

For T5^{XR}, remove bottom three screws and attach bag by reinstalling screws. Tighten screws.

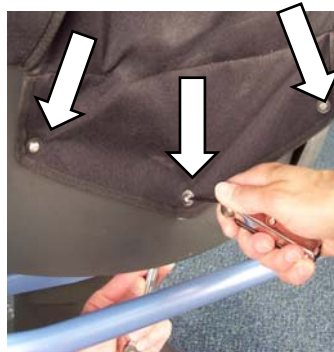



Transforming Lives®


How to Use the Accessory Bag (Rev A)

Make sure bottom of bag is flat and that it hangs behind the swivel release lever.

To store accessories, simply place clipboards, magazines, etc., into upper pocket and WellGrips™ into lower pocket.



To store Leg Stabilizer(s), insert end of Leg Stabilizer into sleeve on bottom of bag, then attach to bag using Velcro straps. Two Leg Stabilizers may be stored this way.

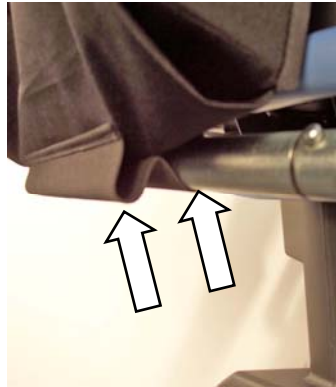

Accessory bag shown with one clipboard, two WellGrips™ and two Leg Stabilizers in a nice tidy package.

