



Transforming Lives®

How to Install the Foot Secure System on a NuStep® T5 / T5^{XR}

Right foot pedal shown in instruction.

Tools Required

10 mm wrench


Accessory Kit Parts

- (2) 50801 Assembly – Ankle Strap
- (2) 50802 Assembly – Toe Strap
- (8) 51215 Bolt-Hex, M6x1x16, Blk Zinc

- Place hex bolt through 1st hole to attach toe buckle strap to inside top hole of footpedal. Place hex bolt through 2nd hole to attach ankle buckle strap to inside bottom hole of footpedal. Attach ladder straps to outside using hex bolts.



- Tighten hex bolts with 10 mm wrench-DO NOT OVERTIGHTEN! Tighten only until hex bolt bottoms out. All straps will be loose with ~ 1/4" (6mm) gap. If overtightened, hex bolt will push threaded fastener out and/or discolor inside of footpedal permanently.

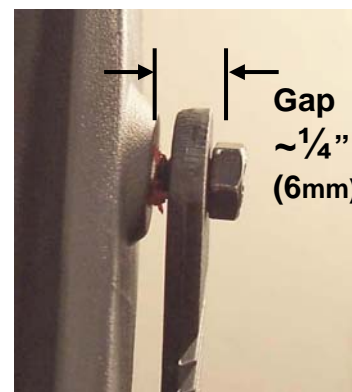

- Feed ladder strap end into buckle strap end.

The completed assembly of the right footpedal is shown far right.



- Complete left footpedal with buckle straps on inside, ladders straps on outside.



Transforming Lives®

How to Use and Store the Footstraps on a NuStep® T5 / T5^{XR}


Right foot pedal shown in instruction.

To Use

Simply place foot on the footpedal and ratchet straps down using black lever.

Tip –

BLACK lever = BUCKLE strap.
RED lever = RELEASE strap.

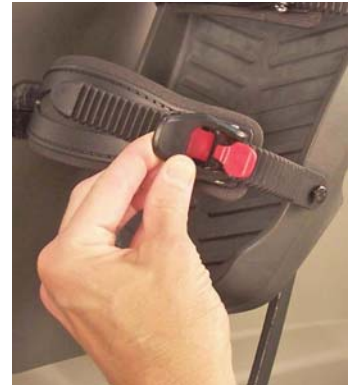

To Store

Simply keep straps buckled in place, or place toe strap behind pedal and ankle strap under heel cup and ratchet till snug.

Do not store straps unbuckled as they will scrape against cover. Do not remove straps from pedals for storage.



To Increase Heel Height

Ankle strap may be tightened down slightly higher than rear heel to add extra heel height.



Materials:

EVA foam padding, PU strapping, rubber, nylon. Latex Free.

Care Guide:

Use an all-purpose non-abrasive, non-bleach cleaner and a soft cloth.

CAUTION

Foot restraint.
User may not be able to remove foot once installed.
To avoid injury, do not over tighten straps.
Supervise use of product.