



Transforming Lives®

How to Install the Torso and Seat Belt Accessory on the NuStep® T5 / T5^{XR}

Tools Required

3mm hex key
4mm hex key
5mm hex key

Accessory Kit Parts

(1) 50792 Assembly, torso belt
(1) 51270 Assembly, retractor, seat belt
(1) 51271 Assembly, buckle, seat belt
(1) 50395 Bracket, seat belt guide
(4) 50399 Screw, BHCS, M8x1.25x22
(2) 50565 Screw, BHCS, M6x1x16

- 1 Remove only the top two screws from back of seat using 3mm hex key.

Install torso belt assembly using same two screws. Tighten screws.



Note: If accessory bag is installed onto seat back, simply install assembly on top of bag using same two screws. Tighten screws.



- 2 Install seat belt retractor assembly onto left side under swivel lever using two 50565 screws and 4mm hex key. Tighten screws.





Transforming Lives®

- 3 Slip 50395 guide bracket through slot between seat and bracket on left side of seat.

Secure guide bracket using two 50399 screws on bottom side of seat. Tighten screws.



- 4 Thread seat belt through guide bracket.




- 5 Slip 51271 buckle assembly through slot between seat and bracket on right side of seat.

Secure assembly using two 50399 screws on bottom side of seat. Tighten screws.



How to Use the Torso and Seat Belt Accessory on the NuStep® T5 / T5^{XR}

To keep user's upper body in good position use torso belt and/or to keep user's lower body in good position use seat belt.

 CAUTION
Body restraint. User may not be able to get off product once installed. Ensure torso belt buckle is accessible in the front. Supervise use of product.

