




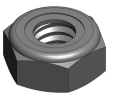


Transforming Lives®




Installation of the Foot Secure System on a NuStep® T4

(Right foot pedal shown throughout instructions)

Foot Secure System Kit Contents

50801 Foot Strap Sets (Ratchet and Ladder) Qty (4)	41583 Screw, 1/4-20 x 1.0 Qty (4)	41166 Washer Qty (2)	41222 Nut, 1/4-20, Locking Qty (4)
			

Required Tools

41582 Drill Template (Included)	Included 41584 Drill Bit, 1/4" (Included)	41585 5/32" Hex Wrench (Included)	Not Included 7/16" Wrench (or Adjustable)
			Power Drill
			Vacuum

Instructions

1. Drill foot strap mounting holes
 - a. Position drill template on right foot pedal (shown) above the pedal link boss.
 - b. Using the included drill bit and a power drill, drill a hole on the inside and outside walls of the pedal. Use the vacuum to collect all falling metal debris. **Prevent debris from falling into the unit between the pedals!**
 - c. Repeat for left side pedal
 - d. Remove aluminum shavings from the holes and clean up surrounding area.






Transforming Lives®


2. Install ratchet and ladder straps
 - a. Place screws into the holes facing outward
 - b. Install (2) inward facing ladder straps onto the screw on the inside of the pedal
 - c. Install nut onto the screw and lightly tighten until the joint is snug. (ability to rotate straps with a light force)
 - d. Install (2) outward facing ratchet straps onto the screw, making sure that the outside strap is installed through the **3rd HOLE (or 1st or 2nd for a larger shoe)**
 - e. Install washer (outboard side only)
 - f. Install nut onto the screw and lightly tighten until the joint is snug. (ability to rotate with a light force)
 - g. Repeat installation for left pedal

* Note: **NO Washer** on inboard side (ladder strap joint)

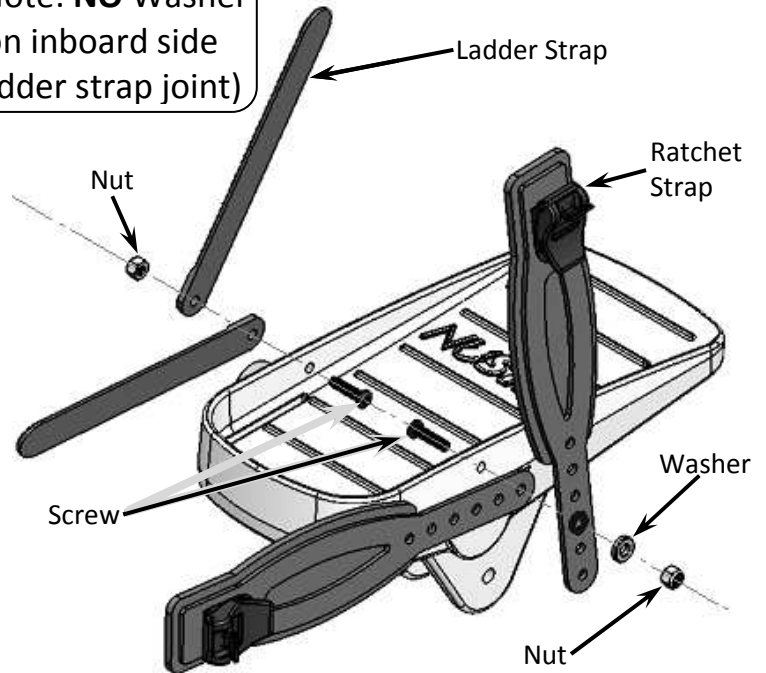

Right Foot Pedal Shown
Ratchet Straps on outside

In Use Position

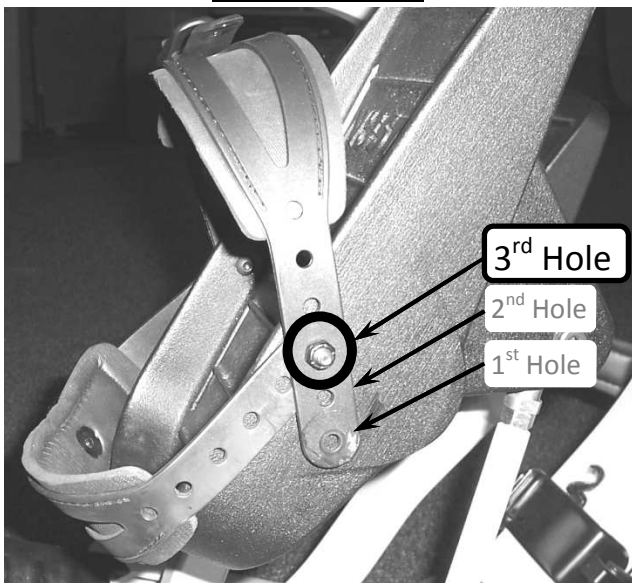

In Stored Position





Transforming Lives®

How to Use the NuStep® T4 Foot Secure System

(Right foot pedal shown)

To Use

Simply place foot on the foot pedal, thread the ladder strap into the ratchet strap buckle, and tighten the heel strap using the ratchet until the desired tightness is reached. Then, position and secure the ankle strap in place using its ratchet straps.

Use of the heel strap will prevent a user's foot from falling off the pedal during the stepping motion if their heel raises, or if a user cannot produce enough dorsal flexion.

Pull RED lever to RELEASE

Push BLACK Lever to BUCKLE



Strap Materials (Latex Free)

- EVA foam padding
- PU strapping
- Rubber
- Nylon

Care Guide

Use an all-purpose non-abrasive, non-bleach cleaner and a soft cloth.

 CAUTION
Foot restraint. User may not be able to remove foot once installed. To avoid injury, do not over tighten straps. Supervise use of product.

NuStep, Inc. • 5111 Venture Drive, Ann Arbor MI 48108 USA • 800-322-4434