# How to Install and Use the NuStep® Leg Stabilizer

English

Français

Deutsch

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Español

Transforming Lives™

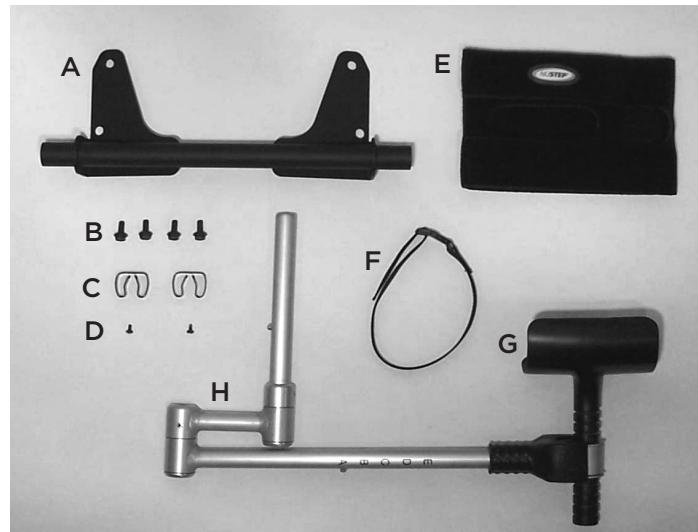
# Parts Diagram

A. 51014 x 1  
Seat mount bracket

B. 51039 x 4  
5/16-18x3/4" Flange  
head cap screws

C. 51041 x 2  
Storage hooks

D. 51042 x 2  
10-24x1/2" Pan head  
screw

E. 51035 x 1  
Leg strap  
(26"/66cm)

F. 51055 x 1  
Storage strap

G. 51033 x 1  
Cuff


H. 51085 x 1  
Stability bar

## Installation Tools Required

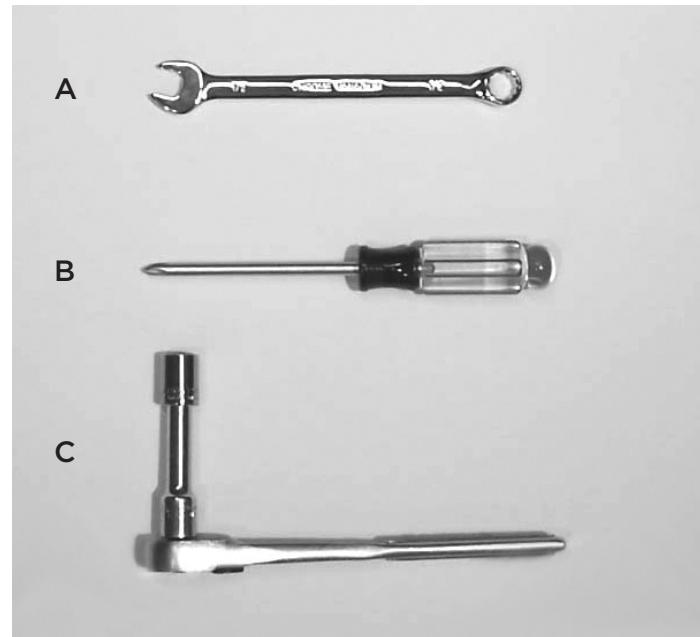A. **1/2" Combination wrench**  
(included)

B. **Phillips screwdriver**

C. **Ratchet, extension, and 1/2" socket**  
(optional)

# How to Install the NuStep® Leg Stabilizer

1. Remove **only the four outer** cap screws and lock-washers on armrest brackets and discard. Do not remove armrests.

2. Position 51014 seat mount bracket over armrest brackets, start four 51039 flange cap screws and tighten to 200 +/- 20 inlbs. (23 +/- 2 Nm)

3. Remove one old shorter screw and install one 51041 storage hook using new longer 51042 screw. Repeat. Don't over tighten screws.

4. Insert 51033 cuff into 51035 strap pocket aligning keyhole shapes. Make sure cuff ends lie flat in pocket.

English

Français

Deutsch

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Español

# How to Use the NuStep® Leg Stabilizer

English

Français

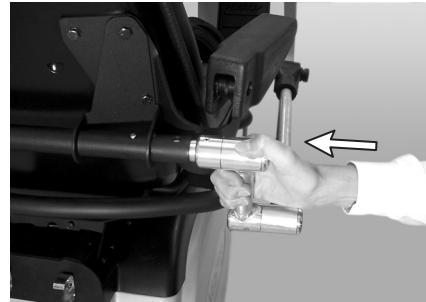
Deutsch

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Español

1. Insert leg stabilizer into left or right side of seat mount bracket until snap button pops into place.


Note, two leg stabilizers can be used at the same time in seat mount bracket to align two legs.

2. Position leg stabilizer near user's leg and depress snap button to adjust for leg position. Cuff should be located just back from knee. There are five positions A - E of adjustment.

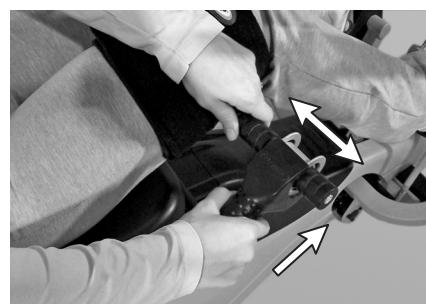3. Drop end of strap in between legs and position flap of cuff over top of leg. Adjust width by pushing grip forward, then slide cuff in and out to get desired leg alignment. There are ten positions 1 - 10 of adjustment.

4. Wrap strap around leg to secure. Strap does not have to be overly tight. A pre-wrap towel etc. may be used against bare skin.

5. Check operation by having user begin NuStepping and make adjustments as required.

## ⚠ CAUTION

To avoid injury, supervise patient during use.

Only align patient's leg to what patient can safely do.

Designed for TRS 4000 use only.

# Additional Notes on Using the NuStep® Leg Stabilizer

- To potentially help move a user's leg into position, install the leg stabilizer with the seat swiveled to the side first, then carefully rotate legs into position.

- For storage, hang product off of storage hooks on rear of machine or...

- ...hang product from armrest using 51055 storage strap.

English

Français

Deutsch

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Español

# Additional Notes on Using the NuStep® Leg Stabilizer

English

- Two leg stabilizers may be used at the same time.

Français

- Storage for two leg stabilizers.

Deutsch

- Having extra accessories around can increase usability. These accessories are available for purchase.

**51035 Leg Strap  
(26"/66 cm)**

**51056 Leg Strap, Large  
(33"/84 cm)**

**51075 Seat Mount  
Bracket Kit**

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Español

# Customer Support

**U.S. and Canadian Customers:**

800-322-4434 or 734-769-3939 x5  
support@nustep.com

**International Customers:**

Please contact your local distributor.

**Materials:**

Nylon, Buna N Rubber, Styrene Butadiene Rubber. Latex Free.

**Strap Care Guide:**

Machine Wash Cold. No Bleach. Tumble Dry Low.

**Product Care Guide:**

Use an all-purpose, non-abrasive, non-bleach cleaner and a soft cloth.

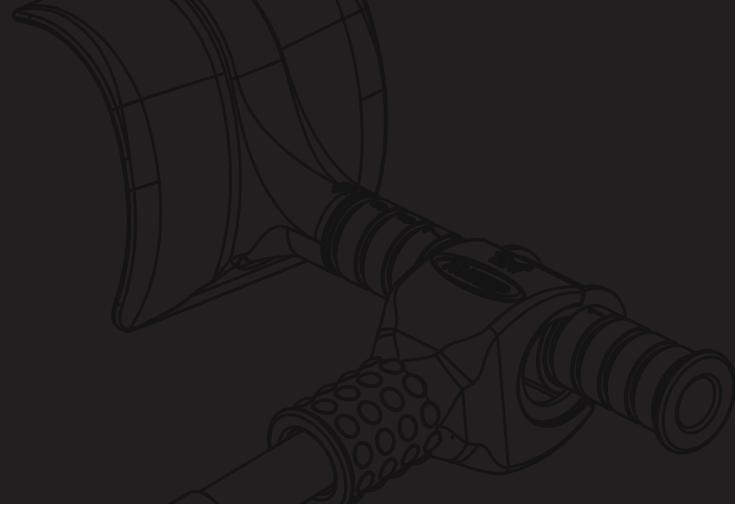
English

Français

Deutsch

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Español

English

Français

Deutsch

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Español

The information in this manual is the most current at the time of printing. Due to our commitment toward continuous improvement, specifications and descriptions subject to change without notice. No part of this manual may be reproduced or transmitted in any form or by any means, electronic or mechanical, for any purpose without the express written permission of NuStep, Inc.

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© Copyright October 2006 by NuStep, Inc.
NuStep® is a registered trademark of NuStep, Inc.
© Copyright octobre 2006 par NuStep, Inc.
© Copyright Oktober 2006 NuStep, Inc.
© Copyright octubre de 2006 por NuStep, Inc.
NuStep® es una marca registrada de NuStep, Inc.