

How to Use the Foot Secure System on a NuStep® T4r



(Left foot pedal shown)

To Use

Place foot on the foot pedal, thread the ladder straps side into the ratchet strap buckles, and tighten using the black ratchet lever until the desired tightness is reached.

The rear strap can be alternately used behind the user's foot to prevent the foot from slipping off the pedal during the stepping motion.

Pull RED lever to Release



Ratchet BLACK lever to tighten



Strap Materials (Latex Free)

- EVA foam padding
- PU strapping
- Rubber
- Nylon

Care Guide

Use an all-purpose non-abrasive, non-bleach cleaner and a soft cloth

⚠ CAUTION
Foot restraint. User may not be able to remove foot once installed. To avoid injury, do not over tighten straps. Supervise use of product.

How to Use the Foot Secure System on a NuStep® T4r



In Use Position



In Stored Position

