## Use of the Seatbelt System on the NuStep® T4r




The Purpose of the seatbelt assembly is to help keep user's body in a proper position on the T4r seat while Nustepping.

- 1 Pull out lap belt from the retractor and clip it into the buckle assembly, as shown.
- 2 Pull adjuster loop on lap belt to take up slack as necessary, as shown. Grasp the Belt on the Retractor side and pull to confirm locking
- 3 To release, Press Red Bottom on Buckle and allow lap belt to retract back into the retractor assembly,



Retractor being clipped into Buckle

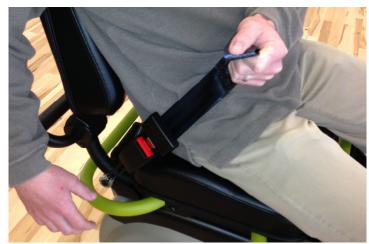

Adjuster loop being pulled to take up slack

## **CAUTION**

Body restraint. User may not be able to unbuckle seatbelt and get off product once lap belt is secured. Supervise use of product.

## Materials:

Polypropylene, Polyester, ABS, Steel

## **Care Guide:**

Clean Belts With Soap & Water Do Not Immerse Do Not Use Bleach.



Red Button on Buckle being pressed to Release Belt